Sonosite PX

User Guide Supplement

Manufacturer

FUJIFILM SonoSite, Inc.

21919 30th Drive SE

Bothell, WA 98021 USA

T: +1-888-482-9449 or +1-425-951-1200

F: +1-425-951-1201

EC Authorized Representative

FUJIFILM SonoSite B.V.

Joop Geesinkweg 140

1114 AB Amsterdam,

The Netherlands

Australia Sponsor

FUJIFILM SonoSite Australasia Pty Ltd

114 Old Pittwater Road

BROOKVALE, NSW, 2100

Australia

Caution Federal (United States) law restricts this device to sale by or on the order of a physician.

SONOSITE and the SONOSITE logo are trademarks and registered trademarks of FUJIFILM SonoSite, Inc. in various jurisdictions. FUJIFILM is a registered trademark of FUJIFILM Corporation in various jurisdictions. Value from Innovation is a trademark of FUJIFILM Holdings America Corporation.

All other trademarks are the property of their respective owners.

Part number: P28610-02

Publication date: October 2020

Copyright © 2020 FUJIFILM SonoSite, Inc. All rights reserved.

Sonosite PX User Guide Supplement

Introduction	1
Document conventions	1
Getting help	2
Getting started	3
Power and battery indicators	3
Selecting a transducer and exam type	4
Configuring the system	4
General settings	4
Entering patient information	4
Using the barcode scanner	5
Measurements and calculations	5
Cleaning and disinfecting	6
Cleaning and disinfecting accessories	6
Safety	7
Electrical safety	7
Battery safety	7
Compatible accessories and peripherals	8
Stand battery specifications	8

Introduction

This user guide supplement provides information on the stand battery, barcode scanner, and the two caliper measurement workflow, now available on the Sonosite PX ultrasound system.

The stand battery is mounted on the Sonosite PX stand and extends the time that the system can run on battery power from up to one hour to up to three hours.

Caution To ensure proper operation of the stand battery and ultrasound system, always have the system battery in the ultrasound system.

Use the USB barcode scanner to quickly enter patient information into the system. The two caliper measurement workflow provides a simplified solution for performing measurements.

Document conventions

The user guide follows these conventions:

- ▶ A  **WARNING** describes precautions necessary to prevent injury or loss of life.

- ▶ A  **Caution** describes precautions necessary to protect the products.
- ▶ A  **Note** provides supplemental information.
- ▶ Numbered and lettered steps must be performed in a specific order.
- ▶ Bulleted lists present information in list format but do not imply a sequence.
- ▶ Single-step procedures begin with ♦.

For a description of labeling symbols that appear on the product, see "Labeling Symbols" in the system user guide.

Getting help

For technical support, please contact FUJIFILM Sonosite as follows:

Phone (U.S. or Canada)	+1 (877) 657-8118
Phone (Outside U.S. or Canada)	+1 (425) 951-1330, or call your local representative
Fax	+1 (425) 951-6700
E-mail	ffss-service@fujifilm.com
Web	www.sonosite.com
Europe Service Center	Main: +31 20 751 2020 English support: +44 14 6234 1151 French support: +33 1 8288 0702 German support: +49 69 8088 4030 Italian support: +39 02 9475 3655 Spanish support: +34 91 123 8451
Asia Service Center	+61 2 9938 8700 ffss-apacme-service@fujifilm.com

Printed in the U.S.

Getting started

Power and battery indicators

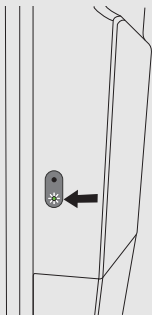
Cautions

- ▶ Connect the system to AC power when the battery charge is low.
- ▶ The stand battery contains battery technology that requires periodic recharging to maintain optimal performance. If left uncharged for months, the battery performance may degrade or become non-functional.
- ▶ Periodically check to make sure that the system and stand battery charges fully. If the batteries fails to charge fully, contact FUJIFILM Sonosite Technical Support (see “[Getting help](#)” on page 2).
- ▶ Use only FUJIFILM Sonosite batteries with the system.

Lights on the stand battery show the power and battery status:

Table 1: Power and battery indicators

Indicator	Status
Lights on the stand battery 	The bottom light turns green when the stand is plugged in and the battery is receiving power. An upper red light indicates that something is wrong with the battery or that the stand is receiving too much power. If the ultrasound system is not docked to the stand, the blue upper light blinks when the stand is plugged in and the battery is charging. When the battery is fully charged, the light remains on but stops blinking.

Selecting a transducer and exam type

The following exam type has been added.

Transducer	Exam type	Imaging mode			
		2D M Mode	Color ^a	PW Doppler ^b	CW Doppler
L19-5 ^c	PIV ^{d, e}	✓	CVD, CPD	✓	

- a. Color Doppler Variance (Var) is available in the cardiac exam only. Color Power Doppler (CPD) is available in all exams except the cardiac exam type. CVD = Color Velocity Doppler
- b. For the cardiac exam types, PW TDI is also available
- c. Needle guide-capable. For more information, refer to *Using CIVCO Products with FUJIFILM Sonosite Systems*.
- d. Available with simultaneous Doppler imaging.
- e. PIV = Peripheral intravenous. The PIV exam type is optimized for PIV procedures.

Configuring the system

To use the barcode scanner with the system, select the following configuration setting.

General settings

To enable the system to query a worklist with the barcode scanner

WARNINGS

- ▶ To avoid damage to the eye, do not stare into the beam. The barcode scanner is a Class 2 laser product.
- ▶ Before using the barcode scanner with patients, verify that any custom programming is working as expected.

You can specify that the system query the worklist for patient information when you scan a patient ID barcode.

- ❖ On the General settings page, select **Barcode auto query**.

Entering patient information

To enter patient information with the barcode scanner, see the following procedure.

Using the barcode scanner

You can use the barcode scanner to directly enter patient information or to enter patient information from a worklist.

To enter patient information with the barcode scanner

- 1 Scan the patient's barcode while the system is in imaging mode.
The patient form appears populated with the patient's information.
- 2 Tap **Scan**.
If you have not ended the previous study, a dialog appears.
- 3 Tap **OK** to scan with the new patient information.

To enter patient information from the worklist with the barcode scanner

You can specify that the system query the worklist for patient information when you scan a patient ID barcode.

- 1 On the General settings page, select **Barcode auto query**.
- 2 Scan the patient's barcode.
Matching patient information appears in the worklist.
- 3 Highlight the patient from the worklist, then tap **Select**.
The patient form populates with the selected patient information.
- 4 Tap **Scan**.
If you have not ended the previous study, a dialog appears.
- 5 Tap **OK** to scan with the new patient information.

Measurements and calculations

Working with calipers

For the following measurements, the calipers now appear as a set when you tap the measurement button instead of sequentially.

- ▶ Distance (2D)
- ▶ MAPSE/TAPSE (M Mode)
- ▶ Angle (2D)
- ▶ HR (M Mode and Doppler)

- Ellipse (2D)
- Time (Doppler)
- Distance Time (M Mode)
- Auto Trace (Doppler)

Use this procedure to activate and position each of the calipers. The active caliper is highlighted.

- 1 Use the touchpad to drag the active caliper to the first point.
- 2 To activate the second caliper, press the **SELECT**/ button.
- 3 Use the touchpad to position the second caliper at the second point.
- 4 As needed, use the **SELECT**/ button and the touchpad to switch between and reposition the calipers.
- 5 Press  to save an image with the measurement.

Cleaning and disinfecting

The following section of the system user guide has been updated.

Cleaning and disinfecting accessories

Clean accessories prior to disinfecting. You can disinfect the exterior surface of accessories using an approved disinfectant. Refer to the cleaners and disinfectants tool available at www.sonosite.com/support/cleaners-disinfectants.

The following procedure applies to most Sonosite PX accessories. For detailed instructions, see the accessory user guide or use the cleaning and disinfecting recommendations in the peripheral manufacturer's instructions.

WARNING To avoid electrical shock, always disconnect the power supply before cleaning the PowerPark.

Caution To avoid damaging the stand battery, do not allow cleaning solution or disinfectant to come in contact with the battery terminals.

To clean and disinfect accessories

- 1 If necessary, unplug the power supply, and detach any cables.
- 2 Clean the exterior surfaces of the accessory using a pre-moistened wipe or soft cloth lightly dampened with an approved cleaner or mid-level disinfectant.
Apply the solution to the cloth rather than the surface.
- 3 With a new wipe or moistened cloth, wipe the surfaces by wiping from clean areas to soiled areas
- 4 Air dry or towel dry with a clean cloth.

Safety

The following safety information has been added to the system user guide.

Electrical safety

WARNINGS To avoid the risk of electrical shock:

- ▶ Do not let any part of the system (including the barcode scanner, power supply, or power supply connector), except for the transducer, touch the patient.
- ▶ Do not disassemble or alter the stand battery.
- ▶ Do not open the battery enclosure on the stand battery and housing.

Electrical safety classification

Ingress protection IP21

Stand battery

Battery safety

WARNING To avoid the risk of fire hazard and burns, do not open, crush, or expose the stand battery to temperatures above 60°C.

Cautions

- ▶ The stand battery contains battery technology that requires periodic recharging to maintain optimal performance. If left uncharged for months, battery performance could degrade or the battery could become non-functional.
- ▶ To prevent the stand battery from draining, do not connect the battery's DC power cord to its own or another stand battery DC input port.

Compatible accessories and peripherals

The following accessories and peripherals have been added to the table in the system user guide.

Table 2: Compatible accessories and peripherals

Description	Maximum cable length
Barcode scanner	4.8 ft/ 1.5 m
Stand battery	—
PowerPark (docking station and stand module)	—

Stand battery specifications

The stand battery comprises twenty-four lithium-ion cells plus electronics, a temperature sensor, and battery contacts.

Dimensions (approximate)

- ▶ **Length:** 19 inches
- ▶ **Width:** 4 inches
- ▶ **Depth:** 2.2 inches
- ▶ **Weight:** 6 lbs

Rating

- ▶ 21.6 VDC, 12000mAh, 259.2Wh

Environmental limits

Operating limits

- ▶ **Temperature:** 0–40°C (32–104°F)
- ▶ **Humidity:** 15–95% R.H.
- ▶ **Atmospheric pressure:** 700–1060 hPa (0.69–1.05 ATM)

Shipping and storage limits

- ▶ **Temperature:** -20–60°C (-4–140°F) for 1 month, -20–45°C (-4–113°F) for 3 months, -20–23°C (-4–73°F) for 1 year.
- ▶ **Humidity:** 15–95% R.H.
- ▶ **Atmospheric pressure:** 500–1060 hPa (0.5–1.05 ATM)

Electrical

- ▶ **Input:** 26.7 VDC, 8.24 A (max 220 W)
- ▶ **Output:** 26.7 VDC, 8.24 A (max 220 W) from power supply or 21.6 VDC, 12,000 mAh, 259.2 Wh from battery

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